


ARROW IAB TO DATASCOPE IABP TUBING ADAPTORS

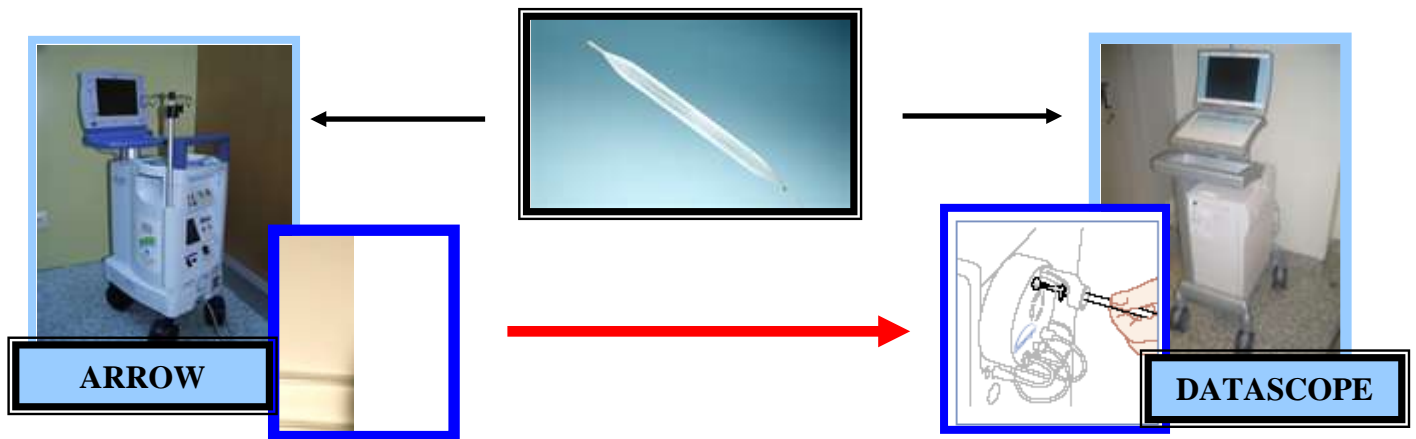

An Arrow IAB can be connected to a Datascope IABP, but the balloon to console connection is different. All Datascope and Arrow IAB insertion kits come with both Datascope and Arrow interconnect tubings.

If a patient is admitted to the CVICU with an Arrow IAB, first ask the transporting personnel if they brought the Datascope tubing with them.

IF YES



Replace the interconnect tubing.

IF NO



Cut the orange, blue, or white Arrow connector off the interconnect tubing. Cut it as close to the adaptor as possible. This is a clean, not sterile procedure.



Then insert a male leurlock, ¼ inch tubing adapter from the envelope below into the end of the cut interconnect tubing. This adaptor will fit into the Datascope IABP.

If the adaptor supply needs to be repleted, please notify Perfusion.